
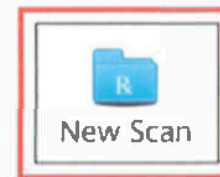


INTRAOURAL SCANNER SETUP INSTRUCTIONS

ITERO ELEMENT

1. Call iTero support (1-800-577-8767) and ask them to have the scans sent to OrthoSelect. Give them the company ID, 14950, and they will instruct you further.
2. When starting a scan, select "New Scan", and enter patient information.
 - a. Choose Case Type: "iRecord"
 - b. Choose Ship To: "OrthoSelect"

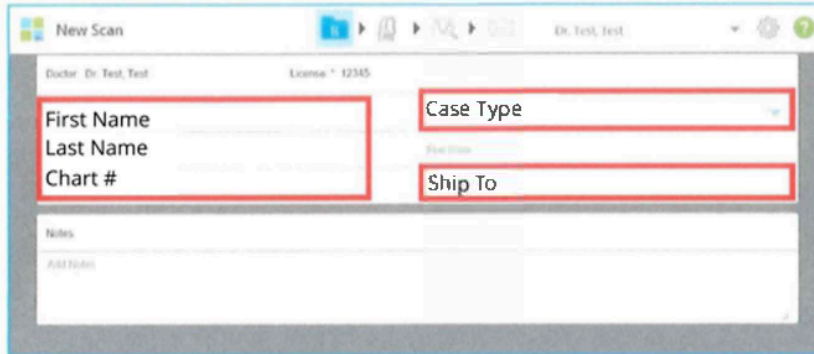

3SHAPE TRIOS

1. Log in to your account at us.3shapecommunicate.com.
2. Click on "Connections".
3. Click on "Add Connection".
4. Type the email address intraoral@myorthoselect.com and click "Find".

CARESTREAM 3600

1. Tell your CareStream representative that you want all of your scans to go through CareStream's system.
2. Provide them with the email address intraoral@myorthoselect.com, which will send a connection invitation to OrthoSelect.



MEDIT I700

1. Log in to www.meditlink.com with the administrator account.
2. Go to Partners on the left menu.
3. Search for the lab that you wish to make a partnership with.
 - a. You can search for partners by their lab name, email address, postal code, or address. Make sure to use a valid email address format: example@mail.com.
4. Locate the lab, and click the Request Partnership button. Check the partner's terms & conditions, and click the Confirm button.
5. You will see that the lab has been added to the Partners > My Partner List with the status reflected as Partnership Pending.
 - a. The "Partnership Pending" status will be changed to "Partnership" when the lab accepts the request.

CEREC/PRIMESCAN

1. When you are in the Portal, click on the(+) in the Recipients tab. It will take you right to the customer connect casecenter.com website.
2. Under My Favorite Recipients, click on Search Recipient
3. Enter in our Zip Code {84003} and click on Start Search
4. Click on the orange(+) to add us to your account.

ALLIEDSTAR

1. Log into your AS Connect account
2. Navigate to the 'My Partner' tab
3. Click 'Add Partner'
4. Search for DIBS AI or support@dibsai.com
5. Click 'Apply'



DEXIS

1. Log into Dexis IS Connect Portal
2. Navigate to 'Partners'
3. Click 'Invite a Partner'
4. Enter our email 'intraoral@myorthoselect.com'
5. Click 'Send'

SHINING 3D DENTAL CLOUD

1. Open the DentalLauncher software.
2. Navigate to the User icon (top right) and select Connections.
3. Click the + New icon.
4. Search for 'OrthoSelect'
5. Click the check mark to submit the connection request.
6. The lab will appear in your "Targets" list after they approve the request.

OTHER SCANNERS

1. For most other scanners, you should be able to get separate upper and lower arch files, which need to be in .stl format.
2. If possible, make sure files are oriented in occlusion.
3. Use "Direct Upload" option on the Rx form to upload .stl files.

